



TIP EXCHANGE AT I/A HANDPIECES

a1 medical

1. PLACE THE SEALING RING WITH THE HELP OF A FORCEPS, E.G. F-3920.

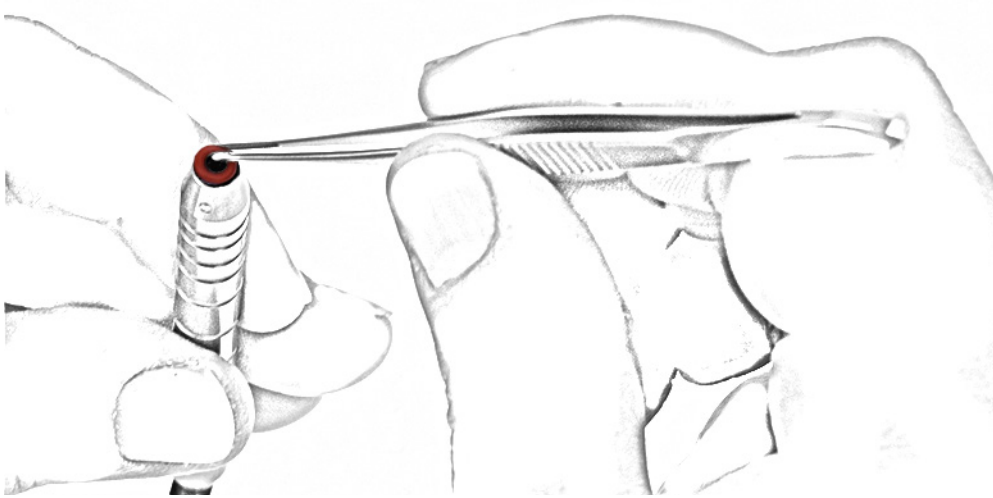


2. HOLDING THE INSTRUMENT UPRIGHT, MAKE SURE THE SEALING RING IS BELOW THE LOCKING PINS AND WELL CENTERED.

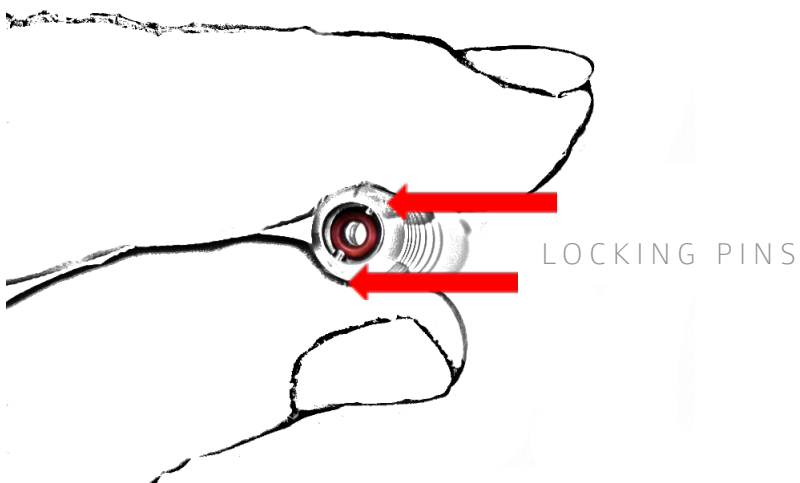

* WITH HANDLES LIKE ITEMS L-1311 OR L-1332
REV: 1 25.07.2016

TIP EXCHANGE AT I/A HANDPIECES

3. PUSH THE INSTRUMENT HEAD ONTO THE HANDLE. WHEN PUSHING, MAKE SURE THAT LOCKING PINS OF HANDLE AND MILLINGS IN INSTRUMENT HEAD ARE ALIGNED.

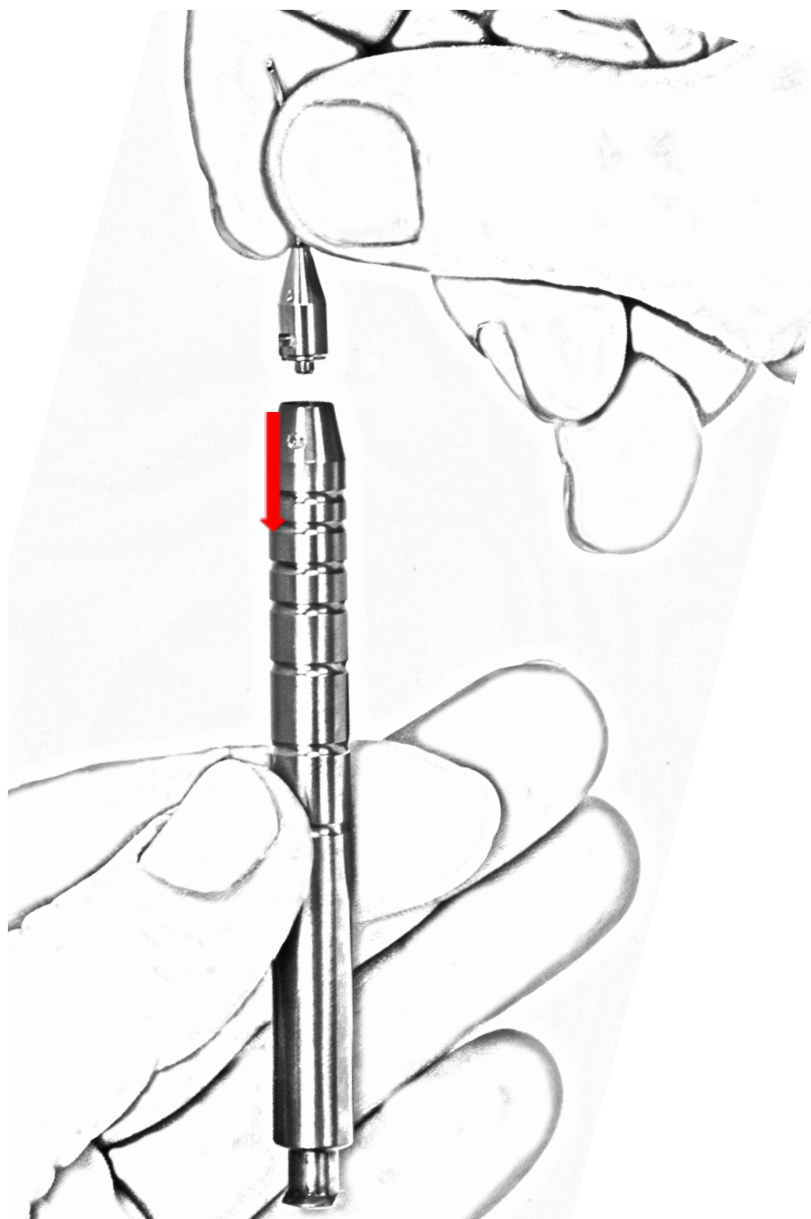


* WITH HANDLES LIKE ITEMS L-1311 OR L-1332
REV: 1 25.07.2016


a1 medical

TIP EXCHANGE AT I/A HANDPIECES

4. WHILE CONTINUING TO PUSH DOWNWARDS, ROTATE INSTRUMENT HEAD CLOCK-WISE (LOOKING FROM HEAD DIRECTION) TO ALIGN POINTS ON HEAD AND HANDLE.

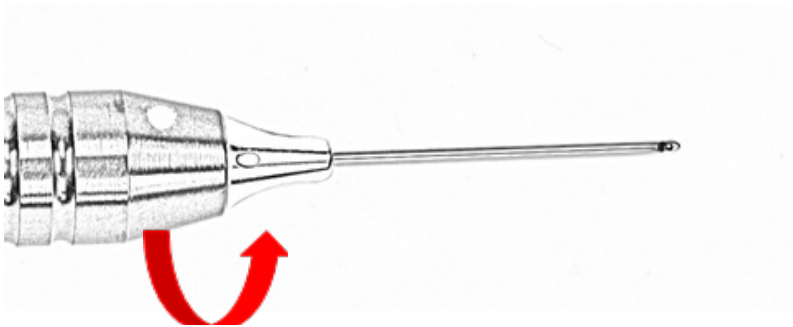


- 5 . WHEN BOTH POINTS ARE ALIGNED, RELEASE HEAD. DONE.

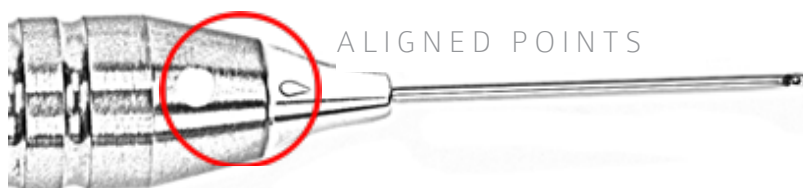

TIP EXCHANGE AT I/A HANDPIECES

a1 medical

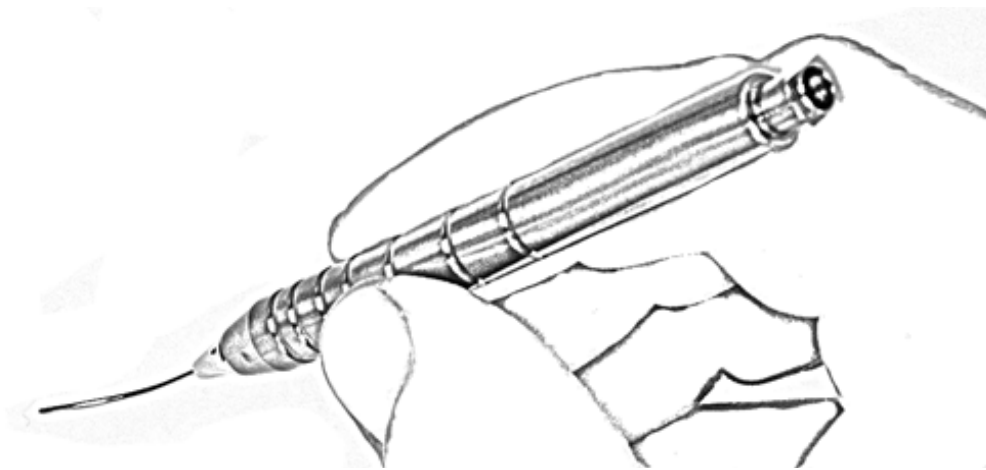

* WITH HANDLES LIKE ITEMS L-1311 OR L-1332
REV: 1 25.07.2016